


ARISE® LAL V-MATT

LOW AIR-LOSS THERAPY SYSTEM FOR VERSACARE® BED FRAME

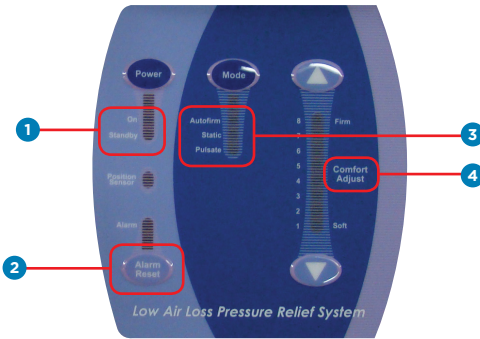

Arise LAL with V-Matt Option – CairRails:

- Provides two inches of additional support along the surface perimeter, helping keep your patient centered and their limbs from resting against the side rail.
- To activate the CairRails, turn the knob towards inflate.
- To let air out of the CairRails, turn the knob towards deflate.



- 1 Power:** Plug in the control unit and the *Standby* light will illuminate. Press the *Power* button.
 - **On:** The control unit will start and the green light will illuminate. Allow three to five minutes for full inflation. After approximately 5 minutes of inactivity the control module will lock out and the power on light begins to flash indicating *Lockout* is enabled. To unlock and make adjustments to the settings press both up and down comfort arrows at the same time to disengage the Lockout function.
 - **Standby:** Surface has lost power.
- 2 Alarm Reset:** Upon power failure the control unit will flash a red *Alarm* light on the keypad and sound an audible signal every two (2) seconds for up to twenty (20) minutes. The power interruption feature keeps the current settings in memory based on the frequency of use and level of battery charge. The *Alarm Reset* will turn off audible and visual alarm.
- 3 Therapy Mode:** To set therapy mode, press Mode until light illuminates next to selected mode.
 - **Autofirm:** Max inflates the surface to aid caregiver in boosts and transfers; automatically turns off after 12 minutes.
 - **Therapy:** Low air-loss therapy.
 - **Pulsate:** Initiates pulsation therapy.
- 4 Comfort Adjust:** Default setting is 5. Adjust firmness according to patient weight; the heavier the patient, the higher the setting.

Please see operator's manual for further information at www.joerns.com.
For further questions, please call your local Agility office or call **1.800.847.7368**.