



○ Harmony® Low Bed – Bed Exit Alarm Quick Reference Guide




WEIGH SCALE (Foot Board Panel – Caregiver)

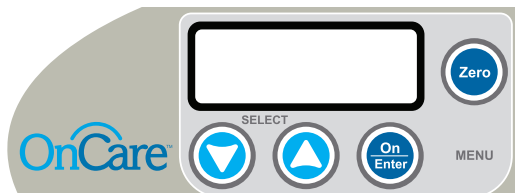



ZERO THE PATIENT WEIGHT SCALE

1. Make bed with all linens and accessories, including necessary equipment.
2. Press and hold  until countdown begins, then release. Display counts down from 5-0 seconds, visually resetting the weight to zero (0.0 lbs).

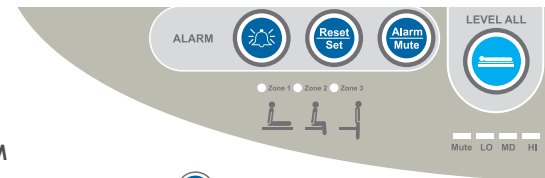
EXIT ALARM (Foot Board Panel – Caregiver)



SETTING THE BED EXIT ALARM

1. With patient in bed press and release the  button, and scroll through the zones to select the appropriate sensitivity (zone 1, 2 or 3). An indicator light will illuminate next to the selected zone. 
2. Select the appropriate audio volume level by pressing the  button as needed to scroll through the options (low, med and hi). An indicator light will be seen showing the selected level is activated. The bed exit alarm is now set.

MUTING THE ALARM

1. Press and release the  button. The button will illuminate indicating that the audio alarm is off.
2. The mute function will automatically disengage or “un-mute” after 10 minutes. The alarm will return to the setting selected prior to enabling the mute function.

Please see user manual for further operating instructions and www.oncaremedical.com for additional product information. For further questions, please call your local UHS office or call **1.800.847.7368**.