



Harmony® Low Bed Quick Reference Guide



BED FUNCTIONS (Foot Board Panel)



HEAD ANGLE

Press and hold to change the head angle.



BED HEIGHT

Press and hold to move the bed up and down.



KNEE-GATCH

Press and hold to change the knee-gatch angle.



CHAIR

Press and hold to move to chair position.



CONTOUR

Press and release to enable the function (back light will change to green), then the head and knee-gatch function will move in sync when either button is pressed.



LEVEL ALL

Press and hold to synchronize head and knee-gatch movement to horizontal.



TREND

Press and hold either of the trendelenburg buttons to set appropriate angle shown on the digital display.



LIGHT

Press and release to switch under bed light on/off.



BRAKE

Press and release to set brake on/off. When off, white back light will flash.



STEER

Press and release to set one foot end caster to a rotational lock position.



CPR


Press and hold to move the bed to a low level horizontal position.



Harmony® Low Bed Quick Reference Guide



WEIGH SCALE (Foot Board Panel – Caregiver)

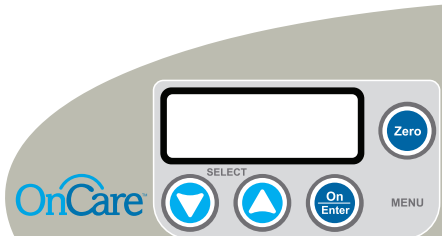

With bed mattress, pillow and linen in-place perform the following set-up prior to placing the patient on the bed.


ZERO THE BED

Press and hold until the countdown begins, then release. Once the bed is zeroed, it is patient ready. Place patient on bed and allow for the displayed weight to remain constant. A symbol "A" under the displayed weight of the new patient will appear. The bed exit alarm can now be set.

WEIGH PATIENT

Press and release any button and patient's weight will be shown on the digital display.


EXIT ALARM (Foot Board Panel – Caregiver)

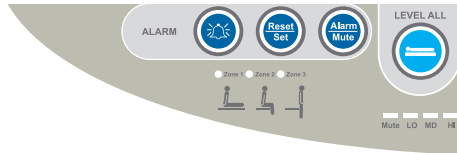

SET SENSITIVITY

Press and release the RESET/SET button multiple times to select and set the appropriate zone sensitivity option: zone 1 (5 kgs.), 2 (5%) or 3 (45%). An indicator light with patient position symbol will display the zone selected.

PATIENT'S SIDE RAIL

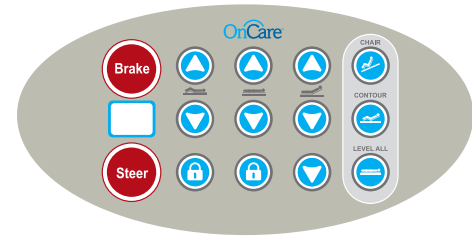

Button groups, symbols and functions are the same as "BED FUNCTIONS (Foot Board Panel)" on side one.



STEER

Press and release to set one foot end caster to a rotational lock position.

Please see user manual for further operating instructions and www.oncaremedical.com for additional product information. For further questions, please call your local UHS office or call **1.800.847.7368**.