


# SENTECH STAGE IV 3000

## LOW AIR-LOSS & ALTERNATING PRESSURE MATTRESS



Universal | Hospital | Services®

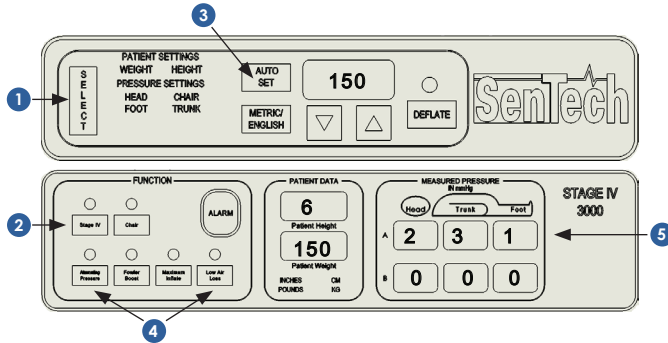

- 1 Lifting the black control panel cover, use SELECT button and up/down arrow keys to set patient's appropriate height and weight.
- 2 Select STAGE IV under the function menu unless using the bedside air chair.
- 3 Press AUTO SET.
- 4 On the main control panel face, select LOW AIR-LOSS and ALTERNATING PRESSURE options, ensuring that the corresponding lights are lit.
- 5 The head, trunk and foot zone pressure settings can be individually adjusted to meet specific patient needs. To set a specific zone, press SELECT (1) until the desired zone setting is illuminated. Use the up/down arrow keys until the desired setting is reached.

NOTE: Pressing the Auto Set button will override these custom settings and return the therapy settings to height/weight based auto settings.

Please see operator's manual for further information at [www.sentechmedical.com](http://www.sentechmedical.com). For further questions, please call your local UHS office or call **1.800.847.7368**.

### Additional Features:

- **Fowler Boost:** Press button when patient is in the fowler position to keep the patient from bottoming out. Disengage when finished.
- **Maximum Inflate:** Press button to create a firm mattress surface. Press again to disengage.

### To deflate mattress for CPR, depending on mattress design:

- Grasp CPR air release on the left side of the mattress, and turn 1/4 turn clockwise **OR**
- Pull the metal CPR latch on the left side of the mattress completely up to the open position.

**Note:** It is important to have approximately 3-4 inches of air between the patient's bottom and the bed frame deck.