




UmanoMedical

QUICK REFERENCE CARD

ook«SNOW» BED

BED OVERVIEW

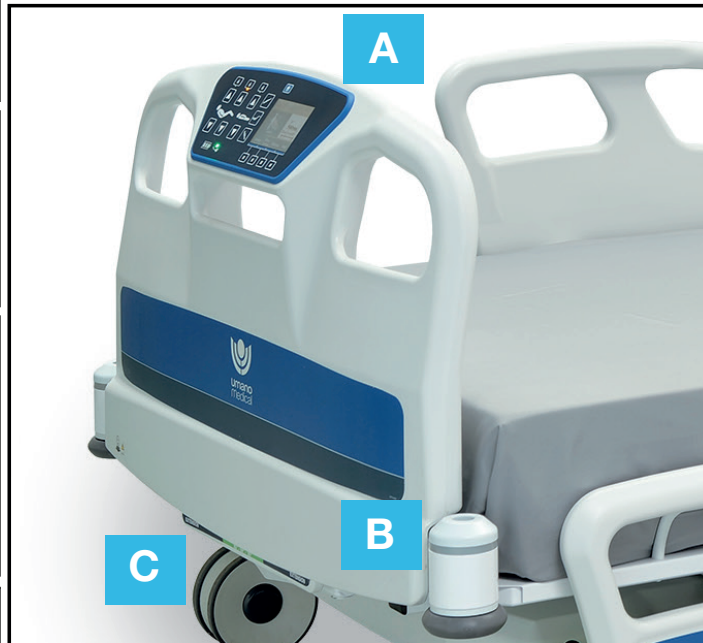

A Main Nurse Control

- Lock out
- Bed movements
- Scale & bed exit
- Battery indicator

B Bed Exit Sideview Lights
(on foot bumpers)

- Green = Armed
- Flashing amber = In alarm
- Not illuminated = Alarm is OFF

C Bed extender (optional)

- Reach & pull on Green handle
- Pull to extend
- Push to retract
- Set to desired length and lock
- Install cushion


D Siderail Controls

- Outside of siderail = nurse controls
- Inside of siderail = patient controls



Siderail controls can be locked at foot end

Reminders

- Ensure bed is plugged to enable scale, bed exit & advanced features



- Apply the brakes

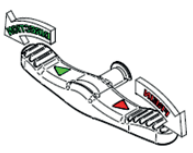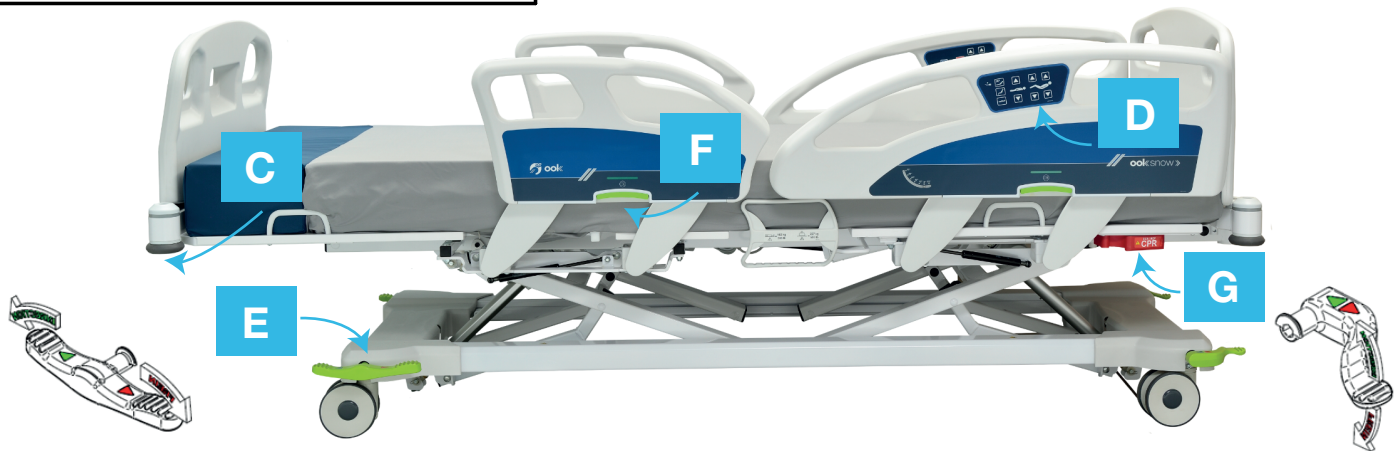E Brake Pedals

- Push Red = Brake ON
- Push Green = Steer ON
- Horizontal = Neutral position (Bed is fully mobile)

F Siderail Release

- Lift green handle UP to unlock
- Lower siderail down

Green window indicates safe status
Red window indicates unlocked siderail

G CPR Release Handle


- Pull red handle
- Completely lower backrest section

Seat section will automatically retract flat.
Press any button to cancel

SCALE & BED EXIT

H HOME Button on the footboard returns the display to HOME SCREEN / MAIN MENU

I NAVIGATION Buttons on footboard will activate commands on MENU

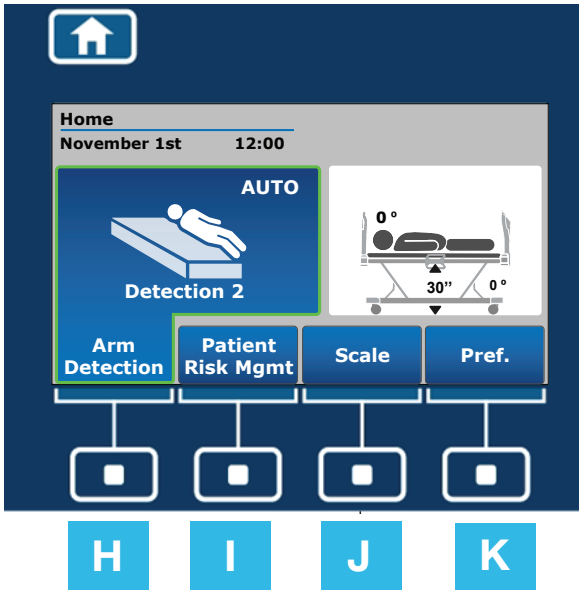

H To arm/disarm BED EXIT system (Regular/Manual setting)


- To arm BED EXIT:
Patient must be in bed to arm bed exit system. From HOME SCREEN, Press 'ARM' **H**
Press 'YES' ('to arm bed exit')
Verify that foot end bumper indicators (optional) are illuminated **green**. When foot bumpers are illuminated **green**, bed exit system is 'ARMED'.
- To disarm BED EXIT (if alarm is triggered) :
Press 'DISARM' (on the HOME SCREEN). This will turn 'OFF' the alarm. Patient must be placed back in bed and alarm must be manually REARMED.

I To change BED EXIT detection level

Please note bed alarm must be 'OFF' to make changes.

To change or set DETECTION LEVEL
From the HOME SCREEN, Press **Patient Risk Mgmt**
Press **Select**

Press **→** to scroll through detection level options and Press 'Select detection' to select desired level.

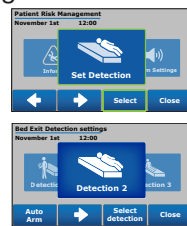

H To arm/disarm BED EXIT system (AUTO Arm) **Auto Arm**

When Auto Arm is activated and patient is detected in bed, the bed exit is armed automatically.

To disarm BED EXIT (if alarm is triggered) :
Press 'DISARM' (which will appear on the HOME SCREEN). This will turn 'OFF' the alarm.

If patient is out of bed :


The system will return to stand-by mode.

If patient is still in bed :

The system will apply a 1-minute care PAUSE to the auto-arm to allow the caregiver to attend to the patient. If after 1 minute patient is still in bed, the system will automatically initiate arming procedure.

J To 'ZERO' out the SCALE

From the HOME SCREEN
Press 'SCALE' button
Press **→** (Press once to scroll to 'Zero')
Press **Select** to 'ZERO'

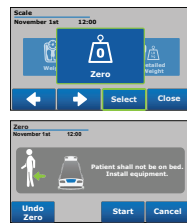

Press 'START' to confirm
Important: Patient should not be on bed during procedure.


I To turn on/off AUTO Arm

From Home Screen, Press **Patient Risk Mgmt**
Press 'Select' to enter Select detection menu
Press 'Auto Arm' to activate or deactivate it



J SCALE - To weigh patient

From the HOME SCREEN, Press 'SCALE' button
Press 'SELECT'
Press 'Weigh' to display weight **Weigh**

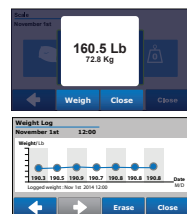

Weight log screen informs of patient logged weight

I To change ALARM settings (Volume and tone)

Alarm must be OFF to change settings.

From the HOME SCREEN

Press 'Patient Risk Mgmt'

Press **→**

Press 'Select' to enter Alarm Settings menu. Make desired changes and press 'OK' to save changes.

K Preference menu

Lights configuration, Lockouts, ID, Auto-contour, Minimum height adjustment, Date/time, Languages, Maintenance

For immediate assistance, please contact your local account executive.