



umano medical

QUICK REFERENCE CARD

ook«snow» ALL BED

### BED OVERVIEW

**A Main nurse controls**

- Lockouts
- Bed movements
- Scale & bed exit
- Battery indicator

**B Bed exit side view lights (on foot bumpers)**

- **Green** = Bed exit system is armed
- Flashing **amber** = In alarm
- Not illuminated = System not in use

**C Bed extender**

- Reach & pull on **green** handle
- Pull to extend & push to retract
- Set to desired length and lock
- Install cushion


**D Infinite™ Width Expansion System**

- Pull out crank and turn to expand or to narrow patient surface (Crank will retract automatically)


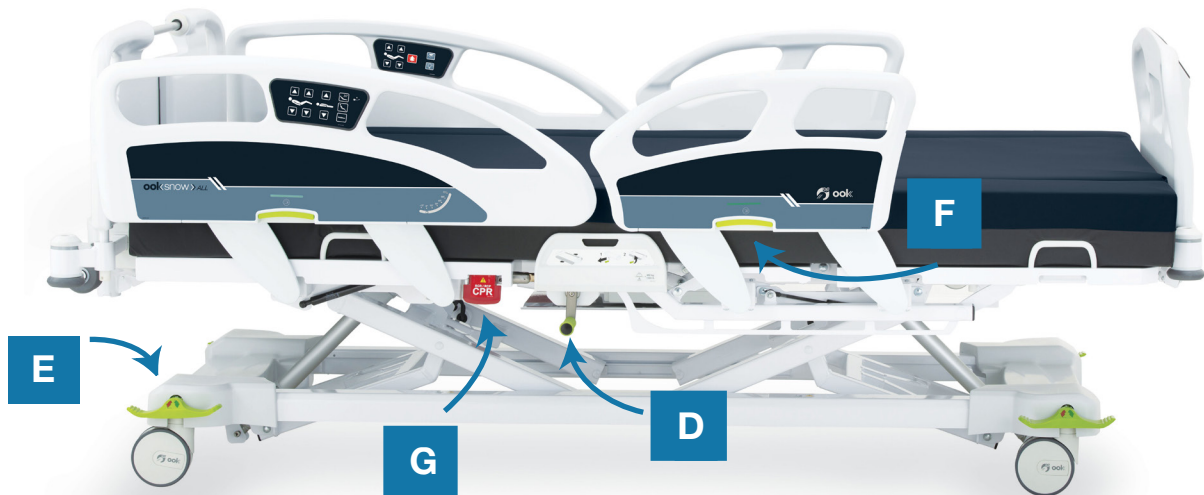


**REMINDERS**

Ensure bed is plugged in to enable scale, bed exit & advanced features



Apply the brakes

**E Brake pedals**

- Push **red** = Brake ON
- Push **green** = Steer ON
- Horizontal = Neutral position (Bed is fully mobile)

**F Siderail release**

- Lift **green** handle UP to unlock
- Lower siderail down
- Green window indicates safe status*
- Red window indicates unlocked siderail*

**G CPR release handle**

- Pull red handle
- Completely lower backrest section
- Seat section will automatically retract flat.*
- Press any button to cancel*

Please refer to the user manual for detailed instructions, safety measures and warnings.

## SCALE & BED EXIT

**H**

HOME button on the footboard returns the display to HOME SCREEN / MAIN MENU

**H**

NAVIGATION buttons on footboard will activate controls

**H**

**I**

**J**

**K**

**H To arm/disarm BED EXIT system (regular/manual setting)**

① To arm BED EXIT  
 Patient must be in bed to arm bed exit system. From HOME SCREEN  
 Press "ARM" **H**  
 Press "YES" (To arm bed exit)  
 Verify that foot end bumper side view lights (optional) are illuminated **green**. When they are, bed exit system is "ARMED".

② To disarm BED EXIT (if alarm is triggered)  
 Press "DISARM" (on HOME SCREEN). This will turn off the system. Patient must be placed back in bed and system must be manually rearmed.

**I To change BED EXIT detection level**

Bed alarm must be off to make changes.

To change or set DETECTION LEVEL  
 From the HOME SCREEN, Press **Patient Risk Mgmt**  
 Press **Select**  
 Press **→** to scroll through detection level options and press "Select detection" to select desired level.

**H To arm/disarm BED EXIT system (AUTO Arm)** **Auto Arm**

When Auto Arm is activated and patient is detected in the bed, the bed exit is armed automatically.

**To disarm BED EXIT (if alarm is triggered) :**  
 Press "DISARM" (which will appear on the HOME SCREEN). This will turn off the alarm.

**If patient is out of bed :**  
 The system will return to stand-by mode.

**If patient is still in bed :**  
 The system will apply a 1-minute care PAUSE to allow the caregiver to attend to the patient. If after 1 minute the patient is still in bed, the system will automatically initiate arming procedure.

**J To ZERO the SCALE**

From HOME SCREEN  
 Press "SCALE" button  
 Press **→** (Press once to scroll to zero)  
 Press **Select** to zero

Press "START" to confirm  
**IMPORTANT : Patient must not be on bed during procedure.**

**I To turn ON/OFF AUTO Arm**

From HOME SCREEN, Press **Patient Risk Mgmt**  
 Press "SELECT" to enter Select Detection Menu  
 Press "Auto Arm" to activate or deactivate it.

**J SCALE - to weigh patient**

From HOME SCREEN, Press "SCALE"  
 Press "SELECT"  
 Press **Weigh** to display weight

Weight log screen informs of patient's logged weights.

**I To change ALARM settings (volume and tone)**

Alarm must be off to change settings.  
 From the HOME SCREEN  
 Press "Patient Risk Mgmt"  
 Press **→**  
 Press "Select" to enter Alarm Settings menu  
 Make desired changes and press "OK" to save changes.

**K Preference menu**

Width management , Lights configuration, Lockouts, ID, Auto-Contour, Minimum height adjustment, Date/time, Languages and Maintenance.

For immediate assistance, please contact your local account executive.