

# Bari Rehab Platform 3™


## Quick Reference Guide



This card is not meant to replace product user manual. For complete manual, contact Sizewise.

800-814-9389

[sizewise.com](http://sizewise.com)

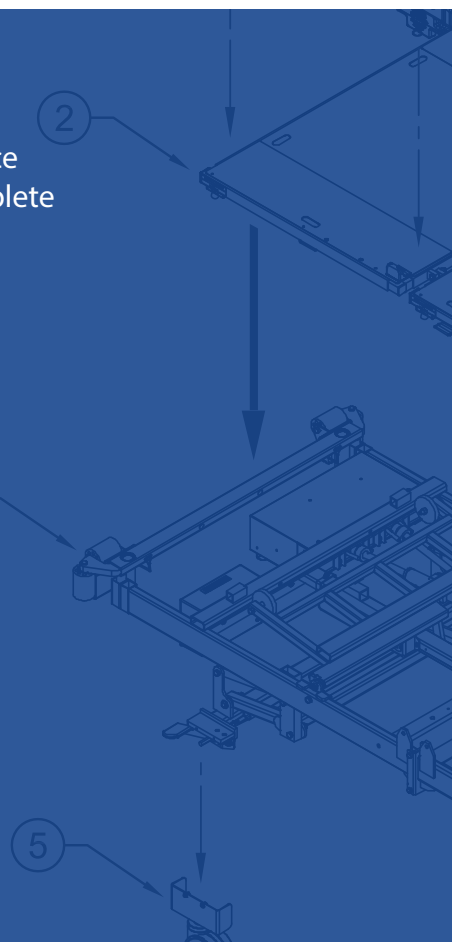

# Components



# Side Rail Installation/Removal



*Slide rail into top deck*



*Install lock pin*

**Removal:** pull out pin and lift rail straight up

# Side Rail Adjustment



*Push to adjust rail*



*Position side rail*

**To adjust:** squeeze knobs while placing rail in desired position

# Casters




Head and Foot casters lock and swivel

- To ENGAGE, press lever down
- To DISENGAGE, raise lever

# Hand Control

Engage brake casters before operating bed

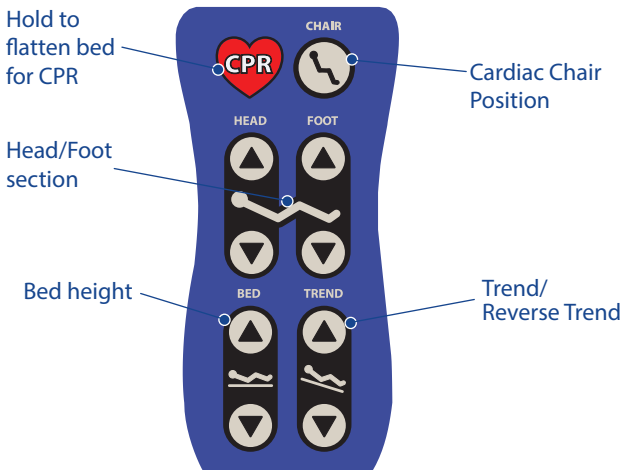

# Expanding/Retracting Bed Deck

Position knob to UNLOCK while pulling out/pushing in section

Expand/Retract all sections on both sides to 48"/39" width



## Troubleshooting

### If bed fails to function:

1. Verify power cord is properly plugged in.
2. Verify main power switch is turned on
3. Verify hand control is plugged into control module at head of bed
4. Verify power source.
5. For further assistance, call 800-814-9389.

## Emergency Hand Crank

*Located under foot section of deck, mounted to motor box cover.*

Remove crank and insert into desired function. Crank either clockwise or counterclockwise to operate. Bed functions located at both foot and head section of bed.



**Foot section:** up/down knees, up/down foot section

**Head section:** up/down torso, up/down head section

## Code Situation

1. Unlock casters and pull bed from wall.
2. Engage brakes, remove headboard, and lower side rails for patient access.
3. Flatten bed by holding CPR button on hand control.
4. If using air surface, activate CPR switch on blower and open CPR valve at head of mattress.
5. Proceed with facility protocols for resuscitation.