


# QUARTET® AUTO IMMERSION

## QUICK REFERENCE GUIDE

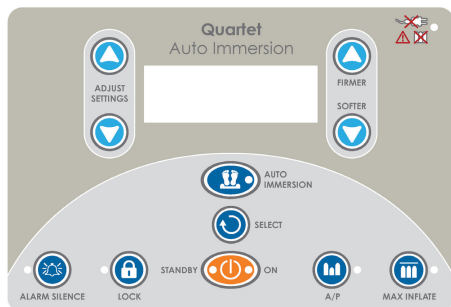

agility™



### LOCK

Press and hold the LOCK button until the LED light turns on, to prevent tampering of settings. Press and hold the LOCK button to turn off.




### POWER

Press the POWER BUTTON to turn on/off the control unit.

### LOW AIR LOSS

The system automatically provides continuous low air loss therapy by supplying air into the low air loss section of the mattress.

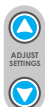

### ADJUST SETTINGS

Press the ADJUST SETTINGS arrows to set various therapy times and patient settings.



### ALTERNATING PRESSURE (A/P)

To set ALTERNATING PRESSURE (A/P) press the A/P button. Use ADJUST SETTINGS arrows to select the desired A/P time, between 1-99 minutes.



### AUTO IMMERSION

With patient on the mattress, press the AUTO IMMERSION button and the blue LED will turn on. The surface will monitor and automatically adjust immersion settings to match the patient weight and position.



### MAX INFLATE

Press the MAX INFLATE button and the mattress will inflate to its maximum volume within 60 seconds (MAX INFLATE automatically deactivates after 15 minutes).



### PATIENT COMFORT CONTROL LEVEL

Press the COMFORT CONTROL arrows to soften or firm the mattress to the patient's preferred comfort level.



### SELECT

The SELECT button cycles between the Pulse and Fowler therapy modes. To select the desired mode, press the SELECT button until that mode is shown on the display. Use the ADJUST SETTINGS arrows to select pulsation frequency and duration options.



### ALARM SILENCE

Press the ALARM SILENCE button until the control unit "beeps" to mute audio alarm.

Please read user manual for further information.

For further questions, contact your local Agility office or call **1.800.847.7368**

©2021 Agility Health, Inc. QUARTETQRG.0721

# QUARTET® AUTO IMMERSION

## QUICK REFERENCE GUIDE



### BED EXIT ALARM

To enable the bed exit alarm, press and hold the ALARM SILENCE button until the control unit “beeps” and follow the instructions on the screen.



### TRANSPORT/BATTERY POWER

The system will convert to battery power when unplugged for transport. Icon displays the percentage of battery capacity. Please keep plugged in whenever possible.

### SAFETY AIR PAD

The 2” safety air pad will remain inflated should the mattress deflate.




### CPR

For CPR or quick deflation, rotate the CPR Valve to open and turn off the control unit.

### HEEL ZONE


1. FIRM
2. MEDIUM FIRM
3. MEDIUM SOFT
4. SOFT

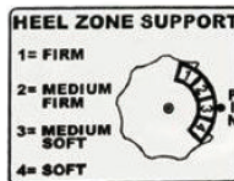

### Heel Zone Regulator

The Heel Zone (bottom four air cells) can be adjusted independently from the rest of the mattress. Turn the Heel zone regulator valve to select the appropriate patient setting.


### LENGTH EXPANSION (both Standard and Bariatric)

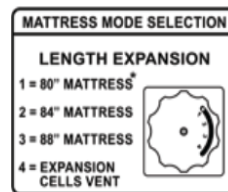

### Length Expansion Regulator

Turn the Length Expansion Regulator valve to select the appropriate length. Select #4 or return to #1 to deflate added air cells.

### WIDTH EXPANSION (Bariatric)

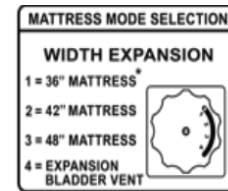

### Width Expansion Regulator

Turn the Width Expansion Regulator valve to select the appropriate width. Select #4 or return to #1 to deflate added air cells.



Please read the user's manual for operating instructions and additional product information.