



- ▶ Adjustable patient platform allows bed to tilt into any angle or chair position for unmatched early mobility therapy
- ▶ Unique ability to accurately measure patient weight bearing ratios for progressive mobility therapy
- ▶ Available options include integrated therapy support surface and bariatric size with motorized drive system

VitalGo Total Lift Bed™: Elevating patient care

The industry-leading VitalGo Total Lift Bed tilts a patient from a 0° flat position to an 82° fully upright position, and any degree in-between. The cutting-edge tilting mechanism, with the assistance of the exclusive Foot Lifter™, raises the bed while keeping the patient's center of gravity to the back and bringing feet to floor level. The first and only bed of its kind, the TLB provides the revolutionary ability to gradually measure and increase weight bearing ratios earlier in treatment that sets an unmatched industry standard for accelerated early mobility therapy. Patient strength can be improved more quickly, potentially reducing length of stay in the health care facility.



Clinical Benefits of Progressive Vertical Positioning:

- Improves respiratory outcomes
- Accelerates post-operative recovery
- Improves blood circulation
- Prevents pressure ulcers and skin breakdown
- Optimizes psychological outcomes
- Provides options for physical therapy and weight bearing



VitalGo Total Lift Bed™ Features and Specifications



How the Weight Bearing Control Works:

The Total Lift Bed features two integrated scales: one located in the sleep deck and one located in the patented Foot Lifter™ footboard. As the bed tilts, a digital screen displays the weight bearing ratio, allowing caregivers to control and gradually increase weight bearing capabilities for accelerated early mobility, improved ambulation and ultimately, optimized patient outcomes.



Patient Applications:

- ▶ Immobile
- ▶ Ventilated
- ▶ Post-Orthopedic Surgery
- ▶ Bariatric
- ▶ Fall Prevention
- ▶ Pulmonary
- ▶ Burn
- ▶ High Risk of Skin Breakdown
- ▶ Post-Operative
- ▶ Early Rehabilitation

Specifications:

	Total Lift Bed	Total Lift Bed Bariatric
Deck Height:		
Low	18.25"	17.7"
High	29.75"	26.5"
Overall Length	83.5" (expands to 90.5")	88" (expands to 93")
Overall Width	40.5"	40" (expands to 52.75")
Sleep Deck	35.5" x 79.3"	35.5" x 79.3" (expands to 47.25" x 85.2")
Weight Capacity	425 lbs.	1,000 lbs.
Tilt Adjustment	0°-82°	0°-82°



Pictured: Bariatric Option with Motorized Drive System

Electronics:

Standard Approvals: EN/IEC 60601-1-2 and 60601-2-38

USA: 120V AC, 60Hz

Visit www.oncaremedical.com to learn more.

OnCare™ VitalGo Total Lift Bed™ and OnCare™ VitalGo Foot Lifter™ are proud offerings of Agiliti Health, Inc. Total Lift Bed™ and Foot Lifter™ are trademarks of VitalGo Systems, Inc.

CORPORATE OFFICE

6625 West 78th Street, Suite 300, Minneapolis, MN, 55439

Toll Free: 800.847.7368 Phone: 952.893.3200 Fax: 952.893.0704 www.agilitihealth.com

©2018 Agiliti Health, Inc. TOTALLIFTBED1218

