

Did you know?



Battery replacement is
easy and convenient
with the
Outlook® Safety Infusion System



B. Efficient.

B | BRAUN
SHARING EXPERTISE

Outlook® Safety Infusion System

Provides Battery Efficiency...



Red to the Positive terminal Black to the Negative terminal



- 12 volt sealed lead-acid battery
- Easy replacement—laptop style
 1. Remove two screws to remove door
 2. Replace battery
 3. Reposition two screws to close door
- Isolated compartment eliminates the need for recertification of pump after battery change
- Inexpensive battery
- Memory is non-volatile and backed up by a lithium battery if battery power is completely depleted

For more information, contact your
B. Braun Representative, Eric Mork,
at 1-800-345-7744, ext. 4069

B | BRAUN
SHARING EXPERTISE
www.bbraunusa.com