


PLEXUS® P2500 AIRE SELECT

LAL/AP SYSTEM



8.0 Operator Control Panel

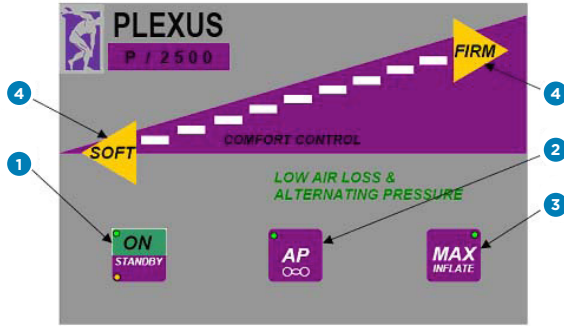

- 1 Press **ON/STANDBY** to turn the system on. A green light will appear.
- 2 To engage alternating pressure therapy, press to illuminate the **AP** button.
- 3 To rapidly create a firm mattress surface for patient ingress, egress or repositioning, press to illuminate the **MAX INFLATE** button. Press again to turn off.
- 4 Mattress pressure/patient comfort can be adjusted by pressing the **SOFT** button to reduce firmness, and the **FIRM** button to increase firmness.

Additional Features:

To deflate mattress for CPR: Disconnect the quick-connect coupling of the mattress hose at the control unit by pushing gray buttons and pulling away from the pump.

Note: Regardless of patient position, it is important to have approximately 3-4 inches of air between the patient's bottom and the bed frame deck.

Please see operator's manual for further information.

For further questions, please call your local Agility office or call **1.800.847.7368**.