


# • ARISE® 1000EX

BARIATRIC LOW AIR-LOSS AND PULSATION THERAPY  
MATTRESS REPLACEMENT



## AIR BLOWER CONTROLS

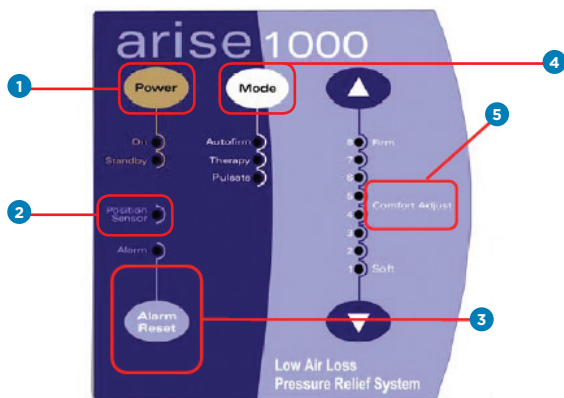

- 1 Power:** Pressing Power turns surface on/off.
  - **On:** Surface has full power; after 12 minutes of inactivity the control module will lock out and the on button will flash green to help prevent unwanted changes to surface – to unlock press the up and down arrows on comfort adjust simultaneously.
  - **Standby:** Surface has lost power.
- 2 Position Sensor:** Automatically activated when head of bed is raised above 30°; pumps more air into sacral region to help prevent bottoming out.
- 3 Alarm Reset:** Alarm will sound when there has been a power interruption; press alarm reset to disable alarm.
- 4 Therapy Modes:** To set therapy mode, press Mode until light illuminates next to selected mode.
  - **Autofirm:** Max inflates the surface to aid caregiver in boosts and transfers; automatically turns off after 12 minutes.
  - **Therapy:** Initiates low air-loss therapy.
  - **Pulsate:** Initiates pulsation therapy.
- 5 Comfort Adjust:** Default setting is 5. Adjust firmness according to comfort and patient weight; the heavier the patient, the higher the setting.

## Additional Features:

**CPR Deflate:** Locate CPR at the point where the air hose connects to the air blower. Squeeze tabs on the side and pull down.


# ARISE® 1000EX

BARIATRIC LOW AIR-LOSS AND PULSATION THERAPY  
MATTRESS REPLACEMENT



## SIDE CONTROLS

- located on right side,  
foot end of mattress

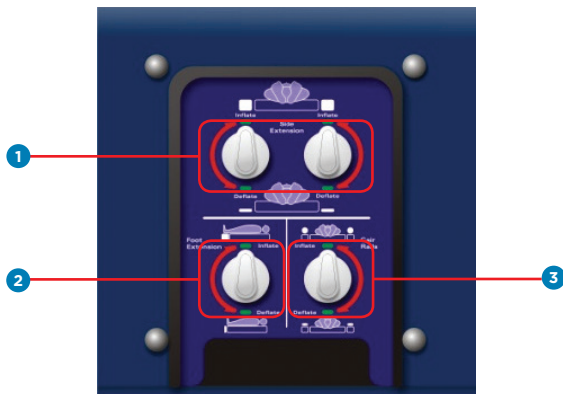

- 1 Inflatable Width Air Bladders:** To activate the width air bladders, turn the white knob to inflate. The air bladder will expand 6" on each side. Air bladders can be inflated simultaneously or independently. To deflate, turn the knob towards deflate.
- 2 Inflatable Length Air Bladder:** To activate the length air bladders, turn the white knob to inflate. The air bladder will expand 8". To deflate, turn the knob towards deflate.
- 3 CairRails:** To activate CairRails to help protect patients from rolling out of bed, turn the knob towards inflate. Two inches of additional support will outline the perimeter. To deflate the CairRails, turn the knob towards deflate.

Please see operator's manual for further information at [www.stryker.com](http://www.stryker.com).  
To learn more about Agiliti solutions please go to [www.Agiliti.com](http://www.Agiliti.com).  
For further questions, please call your local Agiliti office or call **1.800.847.7368**.